15-250

System – Lichtgriff

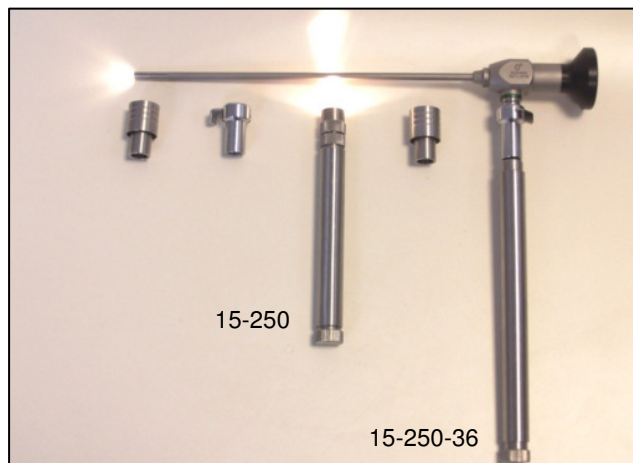
System Light handle

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Highly effective mobile universal light source for Endoscopes with Storz-, Wolf- or ACM-Adapter. Includes high power incandescent lamp or LED and batteries. Suitable especially for mobile endoscopic purposes (e.g. ENT-examinations), Sinusscopes and Nasopharyngoscopes.

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The Faromed System-light-handle provides a minimised size, an incomparable mobility, a high light emission as well as a simple and comfortable handling. The conspicuous intensive and white light is provided by a special lens ended high power lamp. The integrated lens focuses the beam to a concentrated spot into the instrument. The LED models give even up to

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100% more light and a much longer lifetime.

That means the highest maximum mobility and cable-free illumination to the user. Because of the low total weight and the slim designed handle is possible to use it with small Endoscopes (Ø4mm, Ø2,7mm,...) in a very sensitive way.

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After removing the batteries the handle can be sterilised by autoclaving. To switch the light on or off you have just to turn the battery cap a little.

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Our excellent light handle can be used as a torch without an Endoscope in all cases, where good illumination is needed (e.g. mouth torch).

The lamp change can be done easily within seconds by any user. The handle can be attached with standard adapters as used with Fibre optic cables so that Endoscopes with Storz, Wolf or ACM adapters can be used. The adapters are not included and have to be ordered additional.

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Technical data:

Sole light handle	short, 2.5 V	long 3,6V(brighter)
Item code:	15-250	15-250-36
	15-250-LED	15-250-36-LED
Total weight:	61 g	87 g (incl. batteries)
Total length:	approx. 120 mm	approx. 170 mm
Power supply:	2x1.5 V non-recharge-	3x1.2 V rechargeable
	able batteries size AAA	batteries size AAA
Endoscope adapter:	Storz: Item code 12-224	Wolf: Item code 12-222
	ACM: Item code 12-221	
Colour temperature:	ca. 3200K (LED 5600K)	